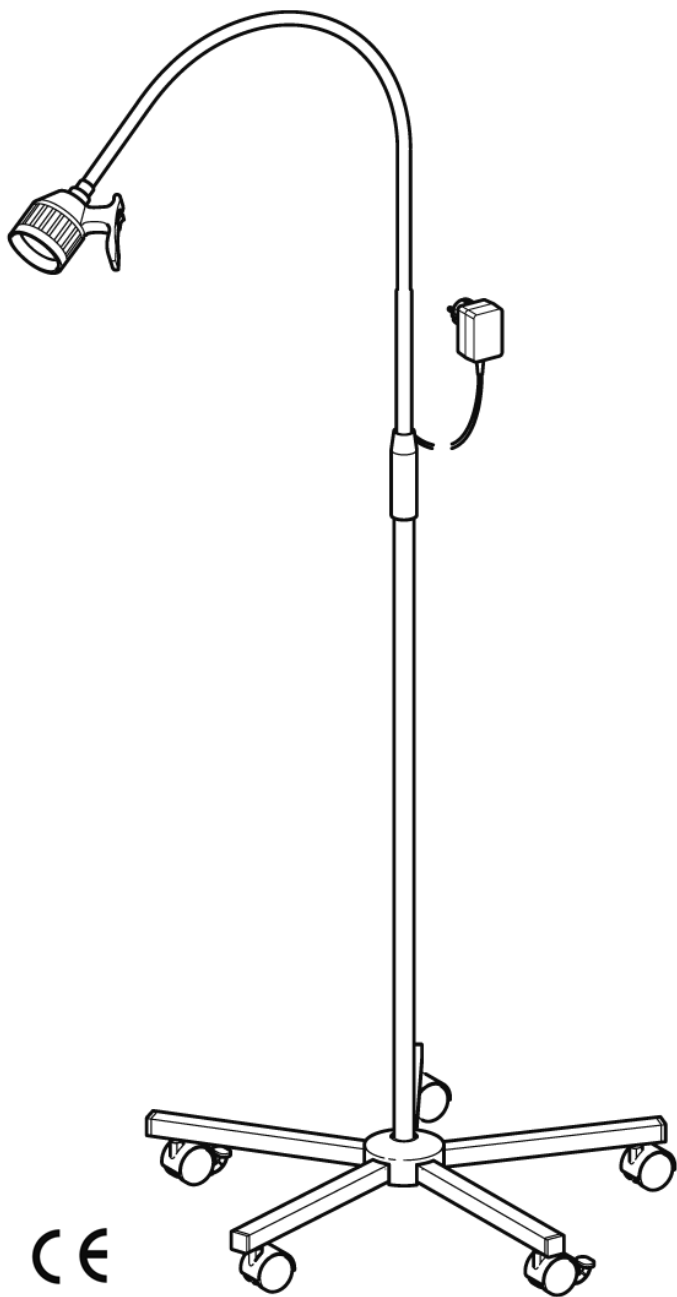
- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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## KaWe MASTERLIGHT® Classic / LED


QM-1-045K

CE

<b>de</b>	Praxisleuchte KaWe MASTERLIGHT® Classic / LED.....	4
<b>en</b>	Examination Lamp KaWe MASTERLIGHT® Classic / LED .....	8
<b>fr</b>	Lampe d'examen KaWe MASTERLIGHT® Classic / LED .....	12
<b>it</b>	Lampada per studio medico KaWe MASTERLIGHT® Classic / LED .....	16
<b>es</b>	Lámpara de exploración KaWe MASTERLIGHT® Classic / LED .....	20
<b>pt</b>	Candeeiro de exame KaWe MASTERLIGHT® Classic / LED .....	24
<b>ru</b>	Светильник смотровой KaWe MASTERLIGHT® Classic / LED.....	28

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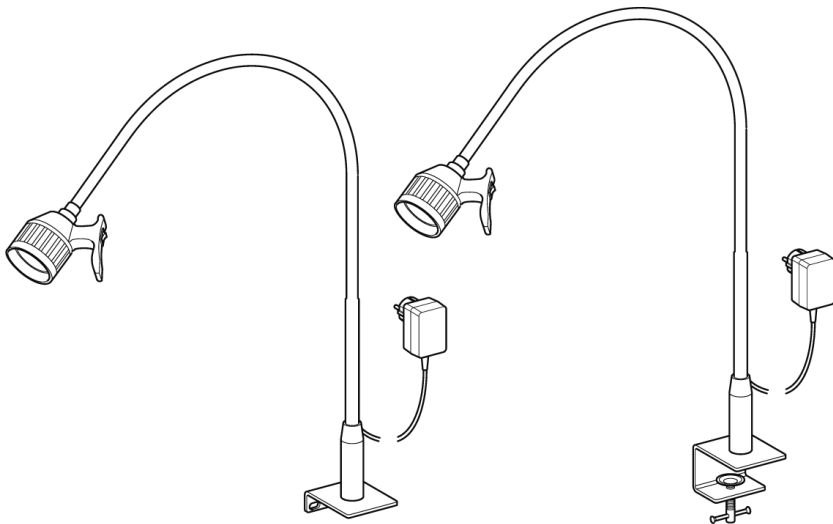
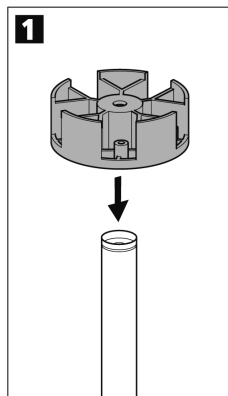
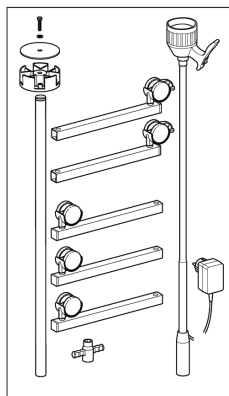
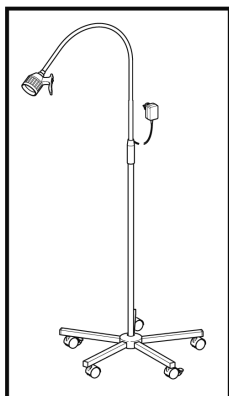
# User's Manual Examination Lamp KaWe MASTERLIGHT® Classic / LED

## EXAMINATION LAMP WITH FIVE-FOOT STAND


Mounting parts


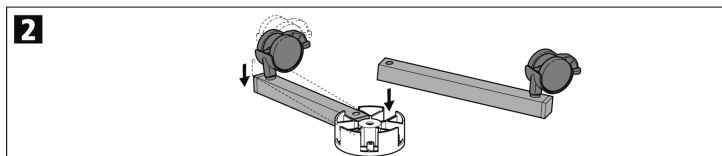
1. Attach the block.

2. Attach two foot pieces with brake rollers.



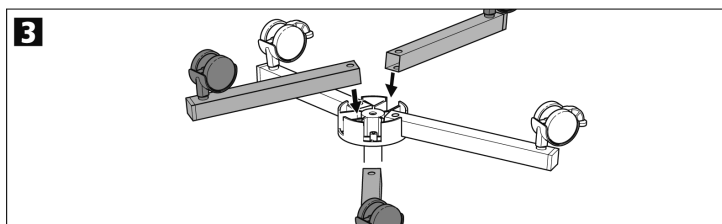
3. Attach three foot pieces without brake rollers.

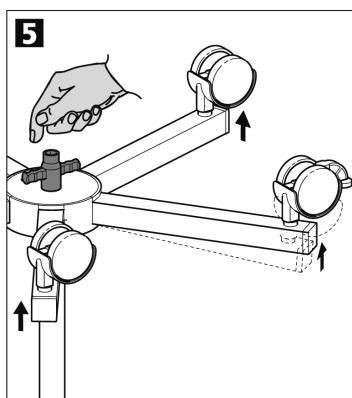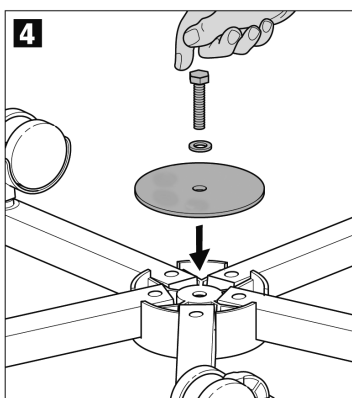4. Screw in hexagon head screws.

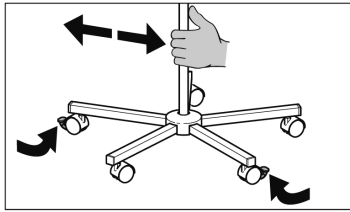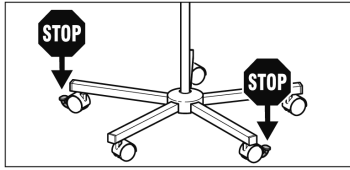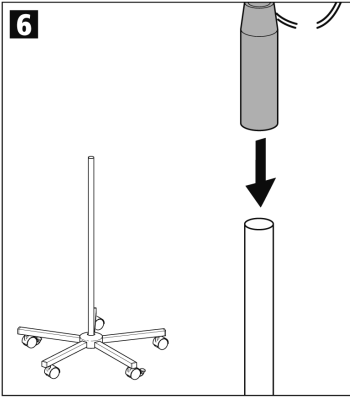5. Tighten the hexagon head screws.




6. Push the upper part of the examination lamp onto the foot base.

Lock the brake rollers. Loosen the brake rollers to move the examination lamp.



### EXAMINATION LAMP – TABLE MOUNTING

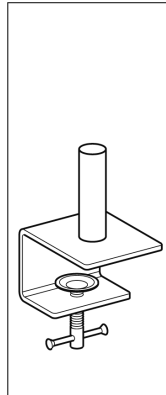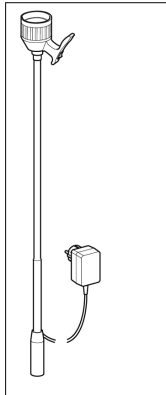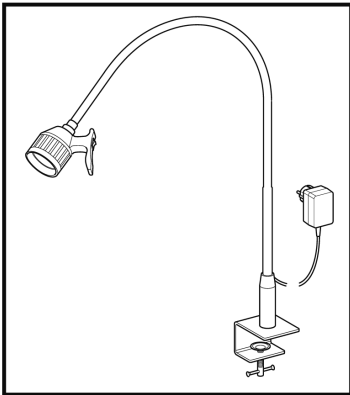Mounting parts

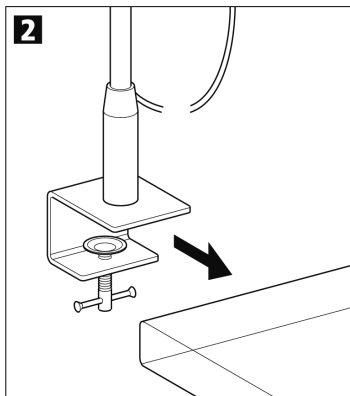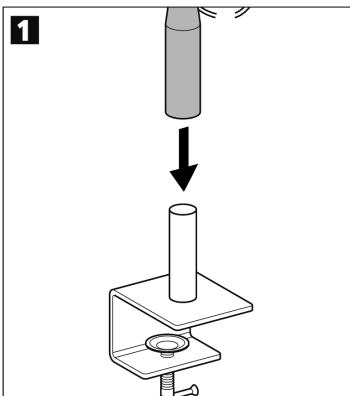1. Push upper part of examination lamp onto table clamp.







2. Fasten examination lamp to the table top.

# EXAMINATION LAMP – WALL MOUNTING

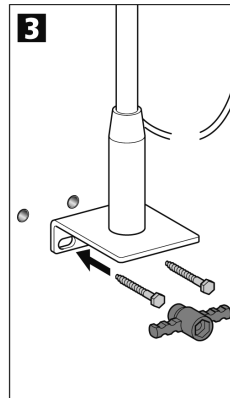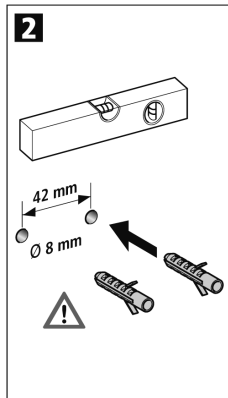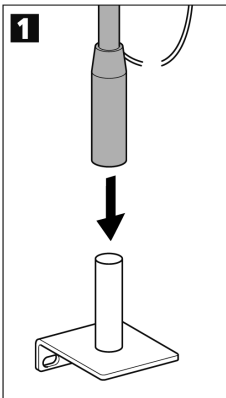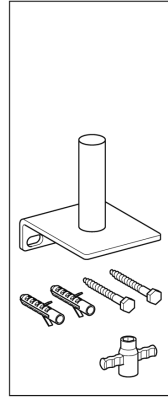
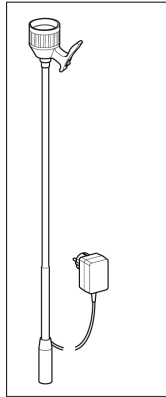
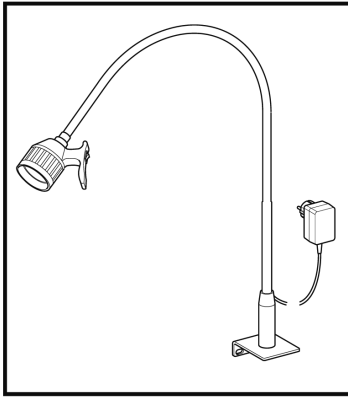
Mounting parts




1. Push upper part of examination lamp onto the wall bracket.

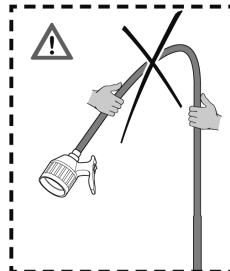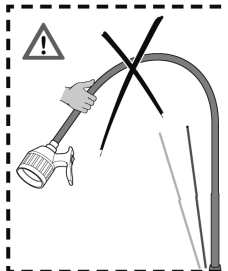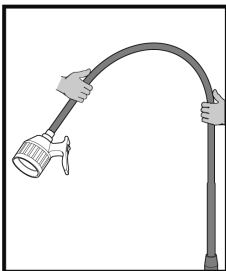2. Use caution when drilling! Check the wall for current lines, gas and water conduits.








3. Affix examination lamp to the wall.

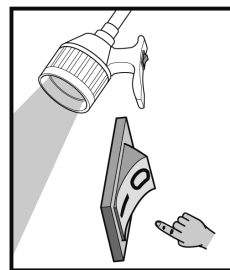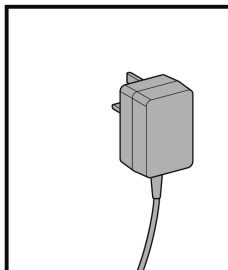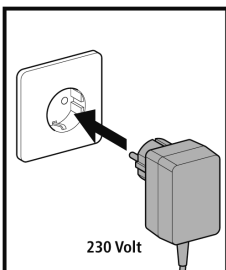Adjust position of light with both hands.

Electric supply  
230 V (EU)  
240 V (GB)

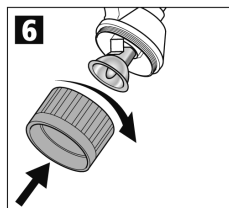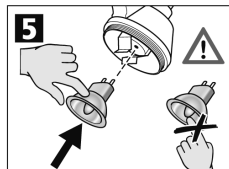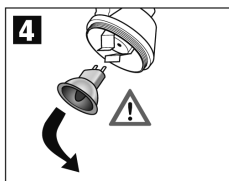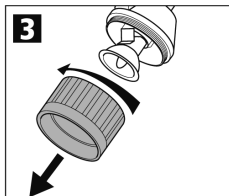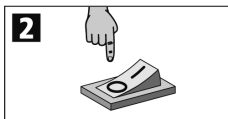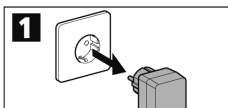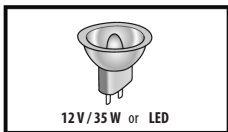Turn ON - I -  
Turn OFF - 0 -

## Replace the bulb

1. Disconnect the power plug.
2. Set switch to - 0 -.
3. Unscrew the ring.
4. Remove defective bulb only after it has cooled down.
5. Plug in the bulb.
6. Unscrew the ring.

**Purpose:** The high-quality flexible examination lamp is to be used in medical practices and in medical facilities.

**Guarantee:** When used properly and with attention to our User's Manual we guarantee this product for 2 years from the date of purchase. For further questions or in case of necessary repair work, please contact your dealer.

**Safety instructions:** 1. Read user's manual and heed it. 2. The examination lamp is only to be used in good order and condition. 3. The examination lamp is only to be used on a flat ground. 4. Protect the mains cable against heat, oil and sharp edges.

**CE Marking:** This KaWe product meets EC Standards for medical products 93/42 EC.

### Technical Data:

Maximum height: ..... approx. 220 cm  
 Upper part – swivel range: ..... approx. 82 cm  
 Diameter tripod stand: ..... approx. 70 cm  
 Weight: ..... approx. 4.4 kg  
 Mains voltage: ..... 230 - 240 Volt, 0.5 A max.  
 Bulb: ..... 12 V / 35 W, 5 A max.  
 LED: ..... 12 V / 10 W, 0.7 A max.  
 Angle of radiation: ..... 8°  
 Colour temperature: ..... approx. 3200° Kelvin  
 Admissible operating temperature: ..... +10° C up to +40° C  
 Relative humidity: ..... 30 % up to +75 %  
 Air pressure: ..... 700 hPa up to 1060 hPa  
 Admissible storage temperature: ..... -40° C up to +80° C  
 Type of protection: ..... IP20

**Cleaning:** 1. The examination lamp is resistant to disinfecting agents, is abrasion-proof and resistant to urine and blood. 2. The examination lamps are to be wiped off with a damp cloth and alcohol-based surface disinfecting agents. 3. Clean soiled rollers in order to avoid static charge. 4. Do not use any acidic substances.

**Maintenance:** After long use, retighten the screws, if required.

**Storage:** Store examination lamp and the mains cable in a dry place.

**Common accessories:** Further information about these products can be obtained in our Internet shop at: [www.kawemed.de](http://www.kawemed.de).

Manufacturer: **KaWe**

**Contacting the manufacturer:** Address or telephone number of the dealer or dial +49-7141-68188-0.

**Removal:** According to the EU Directives this product must be brought to a collection center for electric and electronic waste.

Subject to technical modifications.

**Warning:** To prevent the risk of electric shock, this device must be connected to a circuit breaker

**Warning:** This device is not to be altered or modified in any way.

### Symbol key:

	Manufacturer		Admissible storage temperature
	Date of manufacture		Admissible operating temperature
	Heed the User's Manual		Separate disposal of electric and electronic waste
	Warning!		Meets EC Standards for medical products

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**KIRCHNER & WILHELM** GmbH + Co. KG